
CybGENE High Fidelity Earplugs User Manual

Please read this user manual carefully before using the CybGENE SA-3-1 high fidelity earplugs.

Safety Precautions

1. Please keep the earplugs away from children, for they might get chocked.
2. Please do an allergy test (whether you are allergic to Latex or not) before using the fidelity earplugs. If you are having any allergic reactions, STOP using the earplugs.
3. To achieve the better level of attenuation hearing protection, the earplugs should be worn properly and sealed with your ear canals completely.

Know your SA-3-1 high fidelity earplugs

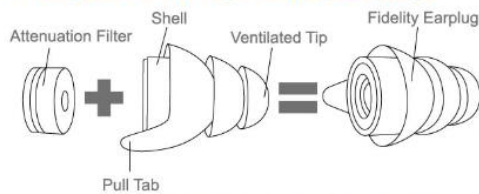How to wear and remove your SA-3-1 high fidelity earplugs

1. Please choose the suitable shells and install them with attenuation filters.
2. Please grip the pull tab and gently push the earplugs into your ear canals until they are sealed and adjust them to a comfortable level for wearing.

3. Please grip the pull tab and gently pull the earplugs away from your ear canals.

How to clean and store your SA-3-1 high fidelity earplugs

1. Please use a damp soapy cloth to clean the shell after each use.
2. Don not clean the earplugs with any abrasive or alcohol.
3. Please keep the earplugs out of direct sunlight, and store them in the bottle provided.

Support

We provide 30 days money back guarantee, 12 months exchange & lifetime support for every customer. Should you have any questions or you need any help, please visit www.cybgene.net or email us at support@cybgene.net.

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